

# Salicule™ Saliva Collection and Test Procedures

## Saliva Collection (Specimen Donor)


1. Unscrew and remove the lids.




2. With the removal of the lids, a straw will extend from the collection vial.



3. Place the straw between your tongue and lips and expectorate saliva till desired volume.

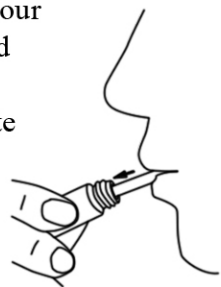

4. Place the lids on top of the straw push down and screw the lids back on to the vial.

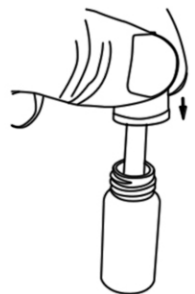

## Discharging Saliva Specimen (Lab Technician)

1. Unscrew and remove the top/smaller lid.



2. Hold the vial upside down and squeeze.

